USER GUIDE 1-8

GUIA DEL USUARIO 9-16

COMODITÀ

Excellence in design and safety

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ROLLING WALKER WITH
ORTHOPEDIC SEAT AND LARGE WHEELS WITH PENEUMATIC TIRES

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COMODITÀ is a company concerned with your comfort, safety and mobility. Our products will simplify your movements, offering support and safety for your daily activities, helping you do what you want, when you want it.

The modern patented design and sturdy construction of the COMODITÀ Uno *all terrain* Rolling Walker make it one of the safest and most stylish walkers in the market. Built to support weights of up to 300 lb, it is tested and approved under international testing standards for walkers and rollators.

The COMODITÀ Uno *all terrain* Rolling Walker is built with the best materials available to ensure the highest safety you can find. In addition, the COMODITÀ Uno *all terrain* Rolling Walker comes with the exclusive COMODITÀ Pisa removable cup holder, cane holder and washable nylon shopping bag, as well an ultra-comfortable seat and backrest.

Our signature "S" shaped back wheel support provides extra room for the feet to move, helping to avoid tripping and falling.

The COMODITÀ Uno *all terrain* Rolling Walker has two 12" aluminum front wheels, with pneumatic tires, making it very easy to use indoors and outdoors. It can also be easily folded for storage or transportation.

Please do not hesitate to contact us through our website

<http://www.comoditaitalia.com/> if you have any doubts, questions or problems. We are here to help!

Other Features:

- Exclusive ultra-comfortable 16" wide orthopedic seat with coccyx relief contour.
- Heavy-duty 300 lb weight capacity.
- 12" aluminum front wheels and 10" in the back, with pneumatic tires.
- Easily adjustable handle height.
- Hand-brake locks for safety while seated.
- Exclusive COMODITÀ Pisa cup holder included
- Our signature "S" shaped back wheel support provides extra wide space between rear wheels to avoid tripping and falling.
- Easily folds for storage and transport.
- Extra wide removable ergonomic backrest.
- Convenient under seat storage bag.

Product Dimensions and Specifications:

- Weight Capacity: 300 lb.
- Recommended User Height: 5'4" to 6'2"
- Overall Height: 34" to 38.5"
- Overall Length: 30.5"
- Overall Width: 25.5"
- Seat Depth: 13.2"
- Seat Width: 16"
- Seat Height from Floor: 22"
- Folded Width: 14"
- Wheel Diameter: 12" in front and 10" in the back
- Unit Weight: 29 lb.
- Distance between handles: 18"
- Recommended tire pressure: 25 PSI

Assembly Instructions:

Assembly your COMODITÀ Uno *all terrain* Rolling Walker is extremely simple.

1. Carefully remove the walker from the shipping carton.
2. Unfold the walker by pushing down the folding brackets on both sides of the rolling walker until they are fully extended.
3. Insert handle bars into the frame of the walker, keeping the brake cables outside the frame.
4. Adjust the height of the handles:
 - Open the handle locking device as shown in the picture below.

- Adjust the height of the handle to the desired position by standing upright behind the walker with your shoulders relaxed and arms hanging at your side. The handgrips should be at a height even with your wrists.
- Close the handle locking device until you hear a "click".

It is that simple. No tools required!

Operating Instructions:

To walk

- Position yourself behind the walker and hold both handles firmly.
- Roll your walker slowly forward so that the rear wheels are a few inches ahead of your body.
- While maintaining a steady and safe roll forward, place your foot in line with the rear wheels.
- Repeat the movement with your other foot.
- To slow down, pull the brake levers up.

To lock and unlock brakes

- To lock the brakes, push down the brake lever until the brakes click into the locked position, as shown below. Do not push it any further than this point.
- To unlock the brakes and resume walking pull up the brake levers until you hear a "click".

To use as a seat

- Make sure your walker is on a flat surface, not on an incline.
- **Lock** the brakes before sitting.
- Make sure your feet rest flat on the ground.

How to install the backrest

Fig 1

Fig 2

Fig 3

- With the locking mechanism positioned at "open", insert the backrest into the plastic support (Fig.1).
- Turn the locking mechanism (Fig.2) to "lock" (Fig 3).
- Do that on both sides.

How to fold your walker

Fig 1

Fig 2

Fig 3
locking device

- Pull up the metal bar located under the seat (Fig 1).
- Close the walker all the way as shown in (Fig 2).
- Lock the walker in the folded position with the plastic device located on the right side of the walker (Fig 3).

Care & Maintenance

Periodic brake adjustment

To adjust the sensitivity of the brake:

- Turn locking nut clockwise to increase brake sensitivity (reduce the gap between the brakepad and the tire).
- Turn locking nut counter-clockwise to reduce sensitivity (increase the gap between the brakepad and the tire).

locking nut

Warning!

- Over adjustment of the brake might prevent the wheels from moving freely.
- Under adjustment of the brake may cause the brake pad not to engage at all.
- ***Do not use the walker if brakes are not working properly.***

Safety Instructions:

Warning! Failure to follow safety instructions may result in serious injury.

- **DO NOT** use this product as a wheelchair.
- **DO NOT** transport someone while sitting on this walker.
- **DO NOT** exceed maximum weight capacity.
- **BRAKES MUST** be in locked position before and sitting on the seat.
- **DO NOT** self-propel the walker while seated.
- **DO NOT** sit on the walker on an incline.
- **INSPECT BRAKES** before each use.
- **ALL WHEELS MUST** be in contact with floor or ground at all times.
- **BACKREST IS NOT** intended to support your full weight.
- **DO NOT** use in escalators, stairs, moving sidewalks or uneven surfaces.
- **BOTH FEET** should rest flat on the ground when sitting.
- **RECOMMEND USER'S HEIGHT** from 5'4" to 6'2".
- Your COMODITA rolling walker should be revised periodically to ensure that the brakes are working perfectly and that all nuts and bolts are tight.

Warranty:

Your COMODITA Uno *all terrain* Rolling Walker is warranted to be free of defects in material and workmanship for a period of one year from the date of purchase from the original owner. This warranty does not cover device failure due to user negligence, misuse of the product, failure to adhere to instructions, or normal wear and tear. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not include any labor or shipping charges. Please contact our customer service at **info@comoditusa.com**.

We will provide a return authorization number. You **will** be required to include a copy of your original receipt.

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for:COMODITA LLC USA